

BioPRYN[®]_{wild}
A Wildlife PREGNANCY TEST

BLOOD SAMPLING SUPPLIES

You can order supplies from Air-Tite. Their 800 number is 1 800 231 7762. Ask for supplies for running the BioPRYN_{wild} test at BioTracking.

Air-Tite Products Co., Inc.
565 Central Drive
Virginia Beach, VA 32454

They should know all of what you need but it is the following:

1. **Part NBT**; Needle holder; get 4 to 6 of these; they are reusable and are onto which you screw the needle. The price is 12 cents each.
2. **Part BD366431**; Vacutainer tubes; these are 7 ML (cc) tubes with a red stopper. The price is \$14.21 per 100. Other tube sizes are available.
3. **Part number NBC 181**; Blood Collecting Needles; 18 gauge by 1 inch. The price is \$19.95 per 100.

BLOOD SAMPLING METHOD

See the BioTracking LLC web site at www.biotracking.com. There is a description for sampling blood from the jugular vein of the female. Learn how to use these sampling supplies.

WHERE AND HOW TO SEND SAMPLES

BioTracking LLC
1150 Alturas Dr Suite 105
Moscow, Idaho 83843

Email:	Biotracking@turbonet.com
Web Site:	www.biotracking.com
Phone:	208-882-9736
Fax:	208-882-1490
Toll Free	866 876 9736

Send without ice, but inside a plastic bag, in a box (tubes break in padded envelopes). Number the tubes with a tube sequence number and the animal number. Also place the two numbers on the Sample Submittal Form. Find this form at www.biotracking.com. Samples are stable and safe even if delayed in shipping. Send UPS or Fed Ex overnight, 2nd day or 3rd day. The post office should deliver Priority Mail within 2 to 4 days.